# Pro-Dye LC®

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## Instructions for use

**Patient target group**

None, as Pro-Dye LC does not come into contact with the human body

**Intended user**

Dental technician

**Composition**

Qualitative data:

Acrylates, methyl methacrylate, camphorquinone, HEMA, TPO, amines, silica, dye

Quantitative data in % G/G:

Acrylates 76 %, activators 18 %, silica 5 %, dye 1 %.

**Intended purpose of the product**

Light-curing one-component coating based on multifunctional acrylates.

**Indications**

The dye varnish Pro-Dye LC is used as a spacer between the dye in the model and the crown.

**Contraindications**

The product contains acrylates. Do not use if acrylate allergy is suspected. If allergic reactions occur, contact/visit a doctor immediately.

**Performance characteristics of the product**

Pro-Dye LC enables an even, scratch-resistant layer of 10µ to be applied to the model die in seconds.

**Specific precautions**

Use under well ventilated fume hood.

**Application**

Shake well before use!

- Apply dye varnish evenly and bubble-free to the model dye using a brush (brush is located in the cap).
  - depending on the UV unit, light-cure the wetted surface for 20-60 sec.
- Observe the polymerization time specifications of the used unit

**Warnhinweise****Restrisiken**

Contains methyl methacrylate  
Contains TPO

**Processing information**

The processing instructions and precautions described in the instructions for use accompanying the product must be observed in all cases. If, despite observance of these instructions, undesirable events should occur, please notify us in any case, giving as detailed a description as possible of the accompanying circumstances.

**Side effects**

Undesirable side effects are not known.

**Interactions**

Interaction of the product with other medical devices, medicinal products or other substances is not known.

**Storage**

Immediately reseal the bottle after removing the varnish, otherwise an unfavorable change in consistency may occur. If necessary, clean the bottle thread with a dry cloth to ensure that the closure is tight.

The Product should be stored between 0° C - 22°C.

**Minimum shelf life**

Do not use after the expiry date.

**Variants**

1 black brush glass bottle, 30ml

 WP5518 - blue

 WP5519 - red

 WP5522 - natural

**Entsorgung**

Unused material should be disposed of appropriately after curing.

**All serious incidents related to the product shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.**

We know how

**dent a pharm**  
Produktionsgesellschaft mbH  
Schusterring 35  
25355 Barmstedt/Germany  
Fon: +(49) 4123/9225-0  
Fax: +(49) 4123/9225-48  
www.dent-a-pharm.de  
info@dent-a-pharm.de

**Willmann & Pein GmbH**  
Schusterring 35  
25355 Barmstedt/Germany  
Fon: +(49) 4123/9228-0  
Fax: +(49) 4123/9228-49  
www.wp-dental.de  
info@wp-dental.de

Attention

Danger

H315 causes skin irritation  
H317 may cause allergic skin reactions  
H319 causes severe eye irritation  
H335 may cause respiratory irritation  
H225 highly flammable liquid and vapor  
H411 toxic to aquatic life with long lasting effects

Made in Germany

21.06.2023

2023-REV-002