# Vision Prophy Powder®

- Seite 2 - de: Gebrauchsanweisung  
Page 3 - en: Instructions for use  
Page 4 - fr: Manuel d'instruction  
Página 5 - es: Instrucciones de uso  
Pagina 6 - it: Istruzioni per l'uso  
Strana 7 - bs: Uputstvo za upotrebu  
Pagină 8 - ro: Instrucțiuni de utilizare  
Sayfa 9 - tr: Kullanma kılavuzu  
Стр. 10 - ru: Инструкция по применению

*We know how*

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**Patientenzielgruppe**

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## Instructions for use

**Patient target group**

Children, adolescents, adults and older patients

**Intended user**

Dentist, dental prophylaxis assistant

**Composition**

Qualitative data:

Sodium hydrogen carbonate, amorphous silica, sweetener and flavouring

**Quantitative data:**

- Sodium hydrogen carbonate 97.3%
- Amorphous silica 1 %
- Sweetener 1.6%
- Flavouring 0.1%

**Intended purpose of the product**

Cleaning the tooth surface

**Indications**

For cleaning only the tooth surface (supragingival)

**Contraindications**

- Patients and practitioners who suffer from chronic bronchitis or asthma must not be treated with or use an air-polishing device under any circumstances. The spray of air and powder could cause respiratory problems.
- Do not use on patients who have upper respiratory tract infections.
- Do not use on patients on a low-salt diet as the powder contains sodium bicarbonate.
- Do not use in case of allergies to the ingredients.
- Do not aim the powder spray at fillings, crowns or bridges as this could damage them.

**Performance characteristics of the product**

Cleans the tooth surface

**Specific precautions**

- Patients and practitioners must wear eye protection
- The practitioner must wear a mask and goggles.
- Use long dental cotton rolls in the patient's mouth for maximum protection.

**Application**

Apply Vaseline to the patient's lips. Fill the powder into the air-polishing device and follow the manufacturer's instructions. Use the large suction cannula on your dental device.

The product may only be used by specialists (dentists).

**Note:**

The maximum powder filling level indicated on the powder container must not be exceeded.

If the powder bottle is left open for too long, the powder may clump and the device may become clogged.

**Warnings Residual risks**

The working instructions and precautionary measures described in the package leaflet accompanying the product must be observed in any case. If an adverse reaction occurs in the patient or other adverse events occur (despite following these instructions), we request that you inform us in every case, describing the accompanying circumstances as precisely as possible.

**Side effects**

No side effects have been reported to date. In case of eye contact, rinse immediately with plenty of water. If you become aware of any adverse effects (even in doubtful cases), please inform us and describe the accompanying circumstances and symptoms as precisely as possible. Risks in case of overdose are not known.

**Interactions**

No interaction of the product and its metabolites with other medical devices, medicinal products or other substances is known of.

**Storage**

The powder should not be stored near acids or heat sources. Store between 0 and 22°C. Protect from moisture.

**Minimum shelf life**

Do not use Vision Prophy Powder after the expiry date.

**Variants**

 WP4913 Black raspberry 1 x 500 g

 WP4914 Mint 1 x 500 g

 WP4933 Lemon 1 x 500 g

**Disposal**

Dispose of unused material according to regulations.

**All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.**

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**Willmann & Pein GmbH**