Secura Varnish®

- Seite 2 - de: Gebrauchsanweisung
Page 3 - en: Instructions for use
Page 4 - fr: Manuel d'instruction
Página 5 - es: Instrucciones de uso
Pagina 6 - it: Istruzioni per l'uso
Strana 7 - bs: Uputstvo za upotrebu
Pagină 8 - ro: Instrucțiuni de utilizare
Sayfa 9 - tr: Kullanma kılavuzu
Стр. 10 - ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentist

Composition

Qualitative data:

Nitrocellulose, triethyl citrate, ethyl acetate

Quantitative data in % G/G:

Nitrocellulose 15

Intended purpose of the product

Sealing varnish

Indications

Secura Varnish is used to seal the surface of glass ionomer cements.

Contraindications

Do not use if there is a known allergy to any ingredient.

Performance characteristics of the product

Self-curing nitrocellulose-based sealing varnish.

Specific precautions

None

Application

After preparation, clean the surfaces to be treated and dry well with warm air. Transfer Secura Varnish drop by drop to the application instrument and apply quickly.

Dry with warm air for 30 seconds and repeat application.

Warnings**Residual risks**

The processing instructions and precautionary measures described in the instructions for use accompanying the product must always be observed. If, despite adherence to these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in any case, describing the accompanying circumstances as precisely as possible

Side effects

No side effects have been reported to date. In individual cases, hypersensitivity to components of the material may occur. Avoid skin contact. In case of accidental contact, wash immediately with water and soap. In case of eye contact, rinse immediately with plenty of water and consult a doctor.

If you become aware of any adverse effects – even in doubtful cases – please inform us and describe the accompanying circumstances and symptoms as precisely as possible.

Risks in case of overdose are not known.

Risks during application: no open flame during application! The solvent is flammable.

Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage

Immediately reseal the bottle after removing the liquid, otherwise an unfavourable change in consistency may occur. The Product should be stored between 0° C - 22°C.

Minimum shelf life

Do not use after the expiry date.

Variants

[REF] WP4099 1 x 10,0 ml liquid

Disposal

Unused material can be disposed of with household waste after curing by evaporation.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

Warning

Danger

Warning

Danger

Made in Germany

H319 Causes serious eye irritation
H224 Extremely flammable liquid and vapour
H336 May cause drowsiness or dizziness

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